

Instructions For Use

Product Name: KN95 Respirator Protective Mask for Health care Use

Type: CD9501B

Structure and composition: The product is composed of mask body, nose clip and mask belt. The mask body has a five-layer structure. Both the inner and outer layers are made of PP spun-bonded non-woven fabric. The intermediate filter layer is 2 layers of PP fusible spray cloth and hot air cotton. The nose clip is made of an aluminum strip. The mask band or ear loops made of a mixture of spandex and polyamide.

Intended use:

This respirators used in a health care setting to prevent health care personnel exposure to pathogenic biological airborne particulates.

Warnings and restrictions on use:

1. Please follow the local regulations and refer to all the information provided by our company to understand the applicability and correct usage of the respirator.
2. Prior to use, wearers must be trained to use the product in accordance with the applicable occupational health and safety standard guidelines.
3. Read the instructions below for fitting the respirator.
4. This product cannot protect against harmful gases and vapors. Do not use in an environment where the wearer would be exposed to gases or vapors.
5. Never use this product for any purpose other than its intended use.

Fitting instructions:

- Do not use with mustaches, beards or other facial hair which can interfere with the good fit between the face and the respirator.
- This respirator can not be modified or repaired at any time.
- This respirators disposable.
- Replace the respirator when it is dirty, damaged or when the user feels the respiratory resistance is obviously increased.
- It also needs to be replaced if the elastic in the mask band has stretched to an extent that proper fit is no longer possible.

To fit the respirator:



With the nose clip side at the top, hold the mask band open with both hands to avoid hand contact with the inside of the mask.



Hold both ends of the mask with your hands, place the mask over your nose and mouth, place the loops around your ears, and adjust to the right position.



Press the nose clip on both sides of the bridge of the nose with your hand to make it fit better.

Storage and transportation:

When the storage temperature is -20℃ to 30℃ and the relative humidity is less than 80%, the storage period of this product is 2 years from the date of production as shown on the carton. Please use the original packaging for storage or transportation.

Manufacturer: Chengde Technology Co.,Ltd

Business address: No.888-1088 Shiji Ave,Longgang City,Zhejiang Province, China

Tel: 0577-59889395

Website: <http://zjlmasks.com/newsdetail/instructions-for-use.html>